



REBOUND

THERAPEUTIC WARMING SYSTEM

FDA Approved Indications for Use:

The ReBounce System is to be used to generate deep heat within body tissues for the treatment of medical conditions such as relief of pain, muscle spasms and joint contractures.

American Medical Association (AMA) Definition of Diathermy:

Creates heat in the soft tissues, due to resistance offered by the tissues by the passage of high frequency electrical currents. The objective of this treatment is to cause vasodilation and/or muscle relaxation, with subsequent pain relief. Diathermy can cause a greater rise in deep tissue temperature than any form of infrared energy.



The ReBounce System is contraindicated for use in patients who have:

- Implanted stimulators such as deep brain stimulators, spinal cord stimulators, and cardiac pacemakers. Also, make certain the implanted leads are not still in place after the stimulator has been removed as this would represent a serious hazard.
- Impaired blood clotting
- Malignancies
- Acute inflammation
- Impaired sensation
- Impaired cognition
- Acute pelvic inflammatory disease
- Women who are pregnant



The ReBounce System is contraindicated in an area over:

- Developing fetus
- Developing gonads
- Metal implants



General Warnings and Precautions:

- Children under the age of 18 should not use the ReBounce System over growth plates.
- It is not recommended for a patient with poorly vascularized tissue(s) to receive a ReBounce treatment.
- Patients with areas of thermal insensitivity should not receive a ReBounce treatment over the affected area.
- Patients with acute joint effusion should not receive a ReBounce treatment.
- Patients should not receive a ReBounce treatment over the ocular and orbital regions (eye area).
- Patients with an intrauterine device should not receive a ReBounce treatment over this area.

ReGear Life Sciences, Inc.

www.regearlife.com

1-888-5REGear (1-888-573-4327)

© 2009 ReGear Life Sciences, Inc. All rights reserved.